*No.1 choice
of hospitals
& mothers*

Swing Maxi™

EN Instructions for use | PL Instrukcja obsługi
TR Kullanım talimatları | CS Návod k použití

Thank you for choosing the Swing Maxi Breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breastpumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. Because you care.

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CS
88-115

Table of Contents

1. Intended Use/Contraindication/Population	5
2. Meaning of symbols	6
3. Important safety information	8
4. Product description	10
5. Cleaning	12
5.1 Before using for the first time and after every use	12
5.2 Before using for the first time and once a day	14
5.3 Cleaning the motor unit	15
6. Preparing for pumping	16
6.1 Mains operation	16
6.2 Battery operation	16
6.3 Assembling the pump set	18
7. Expression	20
7.1 Preparing for single pumping	20
7.2 Pumping	20
7.3 Preparing for hands-free expression with Easy Expression Bustier	22
7.4 Preparing for double pumping	24
8. Storing and thawing breastmilk	26
8.1 Storage	26
8.2 Freezing	26
8.3 Thawing	26
9. Feeding breastmilk	27
10. Troubleshooting	28
11. Warranty/disposal	29
12. Complete care during breastfeeding time	30
13. EMC/Technical description	116
14. Technical specifications	123

1. Intended use/Contraindication/Population

5

The intended use of breastpumps

Breastpumps are intended to be used by lactating women in the hospital or home environment to alleviate the symptoms of engorgement and mastitis by expressing breastmilk from the breast. It is intended to relieve sore and cracked nipples and to bring out flat or inverted nipples. Breastpumps will provide a means to give breastmilk to babies who otherwise cannot feed directly at the breast due to latch-on problems, cleft palate or premature babies.

Intended Patient Population

Adolescent pediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breastpump to express and store their breast milk once they have returned to work, are traveling or are otherwise separated from their baby. Women are often delighted to learn that a breastpump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the suckling of a nursing baby.

There are no contraindications for the Swing Maxi breastpump.

2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury or damage to the breastpump! When used in conjunction with the following words, the warning symbol stands for:

Warning

Can lead to serious injury or death.

Caution

Can lead to minor injury.

Note

Can lead to material damage.

Info

Useful or important information that is not related to safety.

Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates unique GSI Global Trade Item Numbers (GTIN)

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates to follow instructions for use.

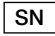
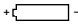
Symbols on the device

This symbol indicates to follow instructions for use.

This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

-  This symbol indicates the manufacturer.
-  This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).
-  This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.
-  This symbol indicates a type BF applied part.
-  This symbol indicates manufacturer's serial number of the device.
-  This symbol indicates the positioning of battery cell.
-  This symbol indicates the protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.
-  This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

Symbols on the mains adapter

-  This symbol indicates that the mains adapter is a class II device.
-  This symbol indicates the compliance with China Compulsory Certification.
-  This symbol indicates that the mains adapter is safety tested.
-  This symbol indicates that the mains adapter is for indoor use only.
-  This symbol indicates the compliance with USA and Canada safety requirements.
-  This symbol indicates the compliance with Federal Communications Commission requirements.
-  This symbol indicates polarity of d.c. power connector.
-  This symbol indicates the compliance with AUS/NZ regulatory requirements (Regulatory compliance mark).
-  This CE-mark indicates the compliance with the low voltage and electromagnetic compatibility directive.
-  This symbol indicates to not dispose the device together with unsorted municipal waste (for EU only). The solid bar symbol indicates that mains adapter is put on the market after 13 August 2005.
-  This symbol indicates the compliance with Japanese safety requirements.
-  This symbol indicates the compliance with energy efficiency requirements.
-  This symbol indicates alternating current.
-  This symbol indicates direct current.

3. Important safety information

8

Failure to follow these instructions/safety information can lead to danger from the device. Subject to technical changes.

Device

Danger of electrocution! Keep the device dry! Never immerse in water or other liquids!

The Swing Maxi breastpump is not heat-resistant: keep away from radiators and open flames.

Do not expose the motor unit to direct sunlight.

Repairs must be performed only by an authorised service agency. Do not repair yourself! No modifications to the device are permitted.

Never use a damaged device. Replace damaged or worn parts.

The operating life of the Swing Maxi breastpump is 250 hours. The service life is two years.

Electrical use

Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.

Keep the mains adapter away from heated surfaces.

Do not reach for any electrical device if it has fallen into water. Unplug immediately.

The breastpump should never be left unattended when plugged into a power source.

Use

Use the Swing Maxi breastpump only for its intended use as described in this manual.

Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.

Do not use the Swing Maxi breastpump while bathing or showering.

Do not use the Swing Maxi breastpump while sleeping or overly drowsy.

This is a single user product. Use by more than one person may present a health risk.

Do not drive whilst hands-free pumping.

Supervision is necessary when the Swing Maxi breastpump is used in the vicinity of children.

Contact your lactation consultant or physician if you experience problems or pain.

Portable and mobile radio frequency communications equipment can affect the Swing Maxi breastpump.

Important: Tray, lids, bags or bottles may become damaged if dropped. Do not use if components become damaged.

4. Product description

10

Spare parts and accessories available for order can be found on page 30

2x PersonalFit™ Breastshield M (24mm)

(for other breast shield sizes see Chapter 12, page 30)

810.7084

2x Freestyle connectors

(breast shield base, membrane, transparent cap)

200.1513

2x bottle

811.0021

2x bottle stand

810.0462

1x Calma Solitaire

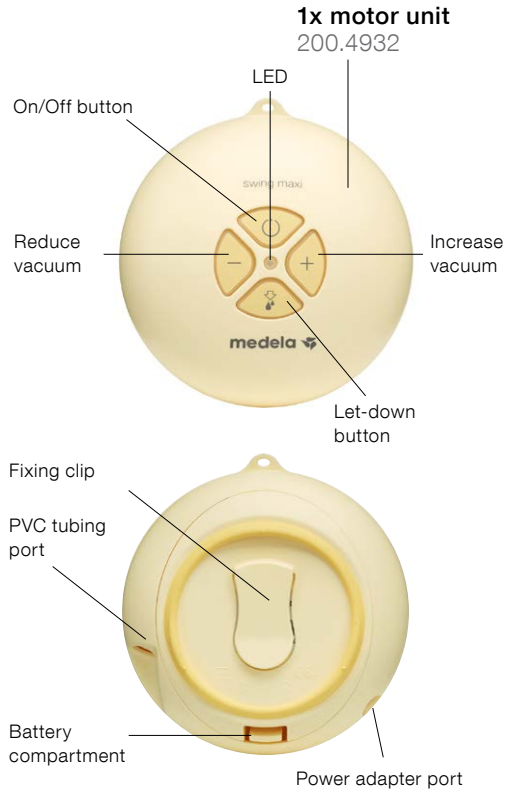
200.3386

2x Calma multi lid

200.2714

5. Cleaning

12

Caution

- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Note

- I The pump set requires no maintenance.
- I Take care not to damage parts of the pump set during cleaning.
- I If the individual components of the pump set are cleaned in a dishwasher parts may be discoloured by food pigments. This has no impact on the function.

5.1 Before using for the first time and after every use

Disassemble the pump set into individual parts.

Rinse all parts with cold, clear water (approx. 20 °C) for 10–15 seconds.

2

Rinse all parts with cold, clear water (approx. 20 °C).

3

Clean all parts with plenty of warm soapy water (approx. 30 °C).

5

Dry with a clean cloth or leave to dry on a clean cloth.

or

As an alternative to steps 2, 3 and 4.

If you clean the individual parts of the pump set in the dishwasher, place them on the top rack or in the cutlery section.

5. Cleaning

14

Caution

- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Note

- I When boiling, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- I Store the pump set in a clean bag/ container until next use. Or store it in clean paper or a cloth.

5.2 Before using for the first time and once a day

More information on page 12, chapter 5.1 steps 2-4

Disassemble the pump set into individual parts.

2c

Use QuickClean* bags in the microwave in accordance with the instructions.

2a

or

2b

or

Cover all parts with water and boil for five minutes.

Use B-Well Steam Steriliser* in accordance with the instructions.

5.3 Cleaning the motor unit

3

Dry with a clean cloth or leave to dry on a clean cloth.

1

Wipe off with a clean, damp cloth.

6. Preparing for pumping

16

Warning

ire 6.1 Mains operation:

Follow the **exact** order of the steps.

- I Only use the mains adapter supplied with the Swing Maxi.
- I Make sure the voltage of the mains adapter is compatible with the power source.

Info

ire 6.2 Battery operation:

The batteries do not need to be removed when the main adapter is used for pumping.

- I Regularly check the condition of the batteries in the motor unit.
- I If Swing Maxi will remain unused for a longer period of time, remove the batteries from the battery compartment.
- I Batteries last for two to three pumping sessions (approx. 1 ½ hours pumping time).

6.1 Mains operation

Plug the mains adapter cord into the motor unit.

6.2 Battery operation

Open the battery compartment on the back in the direction of the arrow.

2

Plug the mains adapter into the mains socket.

2

Insert six AAA Micro 1.5 V batteries.

3

Close the battery compartment.

6. Preparing for pumping

18

Caution

I Wash hands thoroughly (at least one minute) with soap and water before touching the breast and pump parts.

Note

- I Use Medela original accessories only.
- I Check pump set components for wear or damage before use and replace if necessary.
- I All components must be completely dry before use.

Info

I Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.

6.3 Assembling the pump set

1

Carefully insert the light yellow membrane into the base of the breast shield.

4

Screw bottle into connector.

2

Fasten the transparent cap to the connector.

→ Make sure that all three snaps (top and side) are engaged.

3

Push the breast shield onto the connector.

5

Insert the short tubing into the transparent cap as far as it will go.

6

Insert the long tubing into the motor unit.

7. Expression

20

Caution

1 re step 2 (chapter 7.1):

Answer the following questions to find out whether the available breast shield size will fit:

- | Is your nipple moving freely in the tunnel?
- | Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- | Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- | Do you feel the breast emptying all over?
- | Is your nipple pain-free?

If the answer to any of these questions is “No” or if you experience problems or pain during expression, consult your breastfeeding and lactation consultant or midwife.


Note

1 re step 3 (chapter 7.1):

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

- | Wipe the breast with a warm washcloth (do not use alcohol).

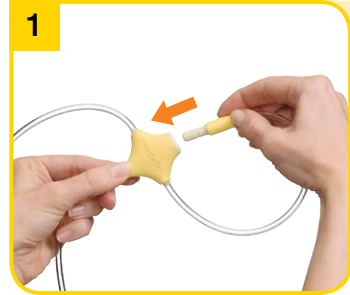7.1 Preparing for single pumping

Insert the unused short tubing into the tubing holder.

7.2 Pumping

Switch on the breast pump with

- The stimulation phase lasts a maximum of two minutes. The stimulation phase will then automatically change to the expression phase.

2

Place the breast shield on your breast so that your nipple is properly **centred** in the tunnel.

3

Hold the breast shield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

2

If your milk starts to flow earlier, press the  button to change to the expression phase.

→ Make sure that the milk is flowing into the bottle properly.

3

The vacuum can be adjusted in every phase. Comfort vacuum: Use  to increase the vacuum until it is slightly unpleasant. Then press  to dial it back minimally.

7. Expression

22

Caution

I Contact your midwife or lactation consultant if you can express only minimal or no milk or if expression is painful.

Note

I Always disconnect the Swing Maxi from the power source after expression.
I Only fill the bottle to the 150 ml mark.

Info

I The Swing Maxi switches off automatically if the pump has been running for 30 minutes without any manipulations.
I Do not kink the tubing during expression.

Overview of LEDs

lights up	On
does not light up	Off
even, regular flashing	Stimulation phase
light constantly on	Expression phase
rapid flashing	Min./Max. vacuum

4

Switch off the breast pump with

7.3 Preparing for hands-free expression with the Easy Expression Bustier*

1

Put on the Easy Expression Bustier and leave the zip open slightly.

5

Use the stand to prevent the bottle from tipping over.

6

Clean in accordance with Chapter 5, page 12.

Close the bottle with a lid.
 → Follow the instructions in Chapter 8 “Storing and thawing breastmilk”.

2

Place the breast shield on your breast under the bustier such that your nipple is **centred** in the tunnel of the breast shield.

3

More information on page 20, Chapter 7.2.

Close the zip and insert the connector onto the breast shield.

*Optional, see Chapter 12, page 30

7. Expression

24

Caution

Important step 2:

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

Info

Double pumping saves time and increases the energy content of the milk. Milk production is increased and maintained long-term.

7.4 Preparing for double pumping

1

Place the breast shield on your breast so that your nipple is properly **centred** in the tunnel.

4

Place the second breast shield on your breast so that your nipple is properly **centred** in the tunnel.

More on page 20,
Chapter 7.2, step 3

2

Hold the breast shield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

3

Switch on the breast pump with

- The stimulation phase lasts a maximum of two minutes. The stimulation phase will then automatically change to the expression phase.

8. Storing and thawing breastmilk

26

8.1 Storage

For healthy, term babies:

	Room temperature	Refrigerator	Refrigerator freezer compartment	Freezer
Freshly expressed breastmilk	Do not store at room temperature	3–5 days at 4 °C	6 months at approx. -16 °C	12 months at approx. -18 °C
Thawed breastmilk (previously frozen)	Do not store	10 hours	Do not refreeze after thawing!	

I Do not store breastmilk in the door of the refrigerator. Instead choose the coolest point in the refrigerator (above the vegetable compartment).

8.2 Freezing

I Never fill bottles or “Pump & Save”^{*} bags more than $\frac{3}{4}$ full to allow space for possible expansion during freezing.

I Label bottles or “Pump & Save” bags with the date of expression.

8.3 Thawing

Warning

Do not thaw frozen breastmilk in a microwave or in a pan of boiling water (danger of burning).

I In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle or “Pump & Save” bag under running warm water (max. 37 °C).

I Gently shake the bottle or “Pump & Save” bag to blend any fat that has separated.

I The B-Well bottle warmer^{*} enables breastmilk and baby food in glass jars to be warmed gently so that there is no loss of vitamins and nutrients.

9. Feeding breastmilk

Info

Medela recommends Calma* for feeding breastmilk.

With Calma

I your baby can feed, pause and breathe regularly.

I milk will only flow when a vacuum is created.

I the milk flow stops if the baby wants to breath or pause while feeding.

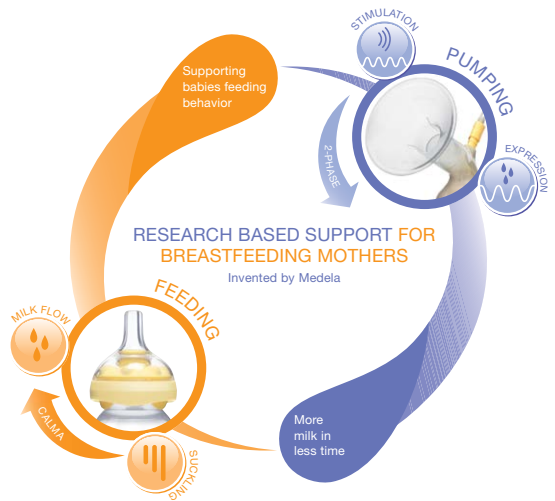
I your baby's natural feeding behaviour will be maintained supporting easy transition back to the breast.

Calma – unique feeding solution for breastmilk.

Detailed information about Calma can be found in the enclosed instructions.

As a result of its comprehensive research work, Medela is one of the first companies in the world to incorporate babies' breastfeeding behaviour into its 2-Phase Expression and Calma innovations. 2-Phase Expression is based on the baby's natural suckling rhythm so that more milk can be expressed in less time. Short, stimulating patterns are followed by longer ones, providing optimal milk flow. The unique Calma breastmilk teat allows the baby to pause and breathe during feeding whilst maintaining the suckling behaviour learned at the breast. Developed for mother and baby for a happy breastfeeding time.

*More information at www.medela.com

10. Troubleshooting

28

Problem	Solution
If the motor is not running	Check presence of mains current. Check the position of the batteries in the battery compartment.
If there is loss of power during battery operation	Replace batteries.
If there is low or no suction	Check all connections. Engage cap correctly at the three connection points. Hold the breast shield, not the connector or bottle, while expressing. Ensure the breast shield forms a complete seal around the breast. During single pumping, insert the unused tubing into the tubing holder. Do not kink the tubing. There should be no water in the tubing. All components must be completely dry.
The pump becomes wet	Unplug the mains adapter from the socket. Switch off the pump. Do not place the pump upside down. Control panel must point upwards. Store the pump in a dry, warm place for 24 hours. Note: Do not expose the pump to direct sunlight. Do not accelerate the drying process by use of heat.
If condensation appears in the tubing	Rinse out the tubing. Shake out any water droplets. Hang up the tubing to air-dry.

11. Warranty / disposal

Warranty

The "International Warranty" brochure contains detailed warranty information.

Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health.

12. Complete care during breastfeeding time

30

The following accessories for the Swing Maxi are available from Medela distribution partners by specifying the product number.

Spare parts

Article number	Product
099.0271	Swing maxi / Freestyle (100-240V; 12V, 50-60Hz, 0,6 A) Euro
099.0272	Swing maxi / Freestyle (100-240V; 12V, 50-60Hz, 0,6 A) UK and US / JP
099.0273	Swing maxi / Freestyle (100-240V; 12V, 50-60Hz, 0,6 A) AUS / NZ
099.0274	Swing maxi / Freestyle (100-240V; 12V, 50-60Hz, 0,6 A) CN
800.0840	Swing Battery cover 5 pcs.
200.1693	Freestyle Connector with valve 5 pcs.
099.0270	Swing Maxi Motor Unit
099.0275	Swing Maxi Tubing

Accessories

Article number	Product
008.0033*	PersonalFit Breastshield S
008.0035*	PersonalFit Breastshield L
008.0036*	PersonalFit Breastshield XL
008.0037*	PersonalFit Breastshield XXL
008.0074	Breastmilk Bottle 150 ml
008.0075	Breastmilk Bottle 250 ml
008.0145	Calma Solitaire
200.1756	Cooling Bag
200.0625	City Style Bag
008.0217	Easy Expression Bustier white S
008.0220	Easy Expression Bustier white M
008.0224	Easy Expression Bustier white L
008.0225	Easy Expression Bustier white XL

Other Medela products are available on the www.medela.com web site.

* If the result of expression is unsatisfactory or expression is painful, contact your midwife or lactation consultant. A different size of PersonalFit™ breast shield may help make expression more comfortable and successful.

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13. EMC/Technical description

The Swing Maxi breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.

Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the electric breastpump Swing Maxi and should be kept at least a distance 1.0 m away from the equipment.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 1)

Electromagnetic emissions		
The electric breastpump Swing Maxi is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Swing Maxi should assure that it is used in such an environment.		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The electric breastpump Swing Maxi uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The electric breastpump Swing Maxi is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

 **Warning** – The electric breastpump Swing Maxi should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the electric breastpump Swing Maxi should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 2)

Electromagnetic immunity			
<p>The electric breastpump Swing Maxi is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Swing Maxi should assure that it is used in such an environment.</p>			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input / output lines	± 2 kV for power supply lines ± 1 kV for input/ output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.

13. EMC/Technical description

<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p><5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s</p>	<p><5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breastpump Swing Maxi requires continued operation during power mains interruptions, it is recommended that the electric breastpump Swing Maxi is powered from an uninterruptible power supply or a battery.</p>
<p>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</p>	<p>3 A/m</p>	<p>30 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>
<p>NOTE U_T is the a.c. mains voltage prior to application of the test level.</p>			

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 4)

Electromagnetic immunity			
The electric breastpump Swing Maxi is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Swing Maxi should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	10 Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the electric breastpump Swing Maxi, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance $d = 0,35\sqrt{P}$</p> <p>$d = 0,35\sqrt{P}$ 80 MHz to 800 MHz $d = 0,7\sqrt{P}$ 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	10 V/m	

13. EMC/Technical description

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electric breastpump Swing Maxi is used exceeds the applicable RF compliance level above, the electric breastpump Swing Maxi should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the electric breastpump Swing Maxi.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 6)

Recommended separation distances between portable and mobile RF communications equipment and the electric breastpump Swing Maxi

The electric breastpump Swing Maxi is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the electric breastpump Swing Maxi can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the electric breastpump Swing Maxi as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = 0.35\sqrt{P}$	80 MHz to 800 MHz $d = 0.35\sqrt{P}$	800 MHz to 2.5 GHz $d = 0.7\sqrt{P}$
0.01	0.04	0.04	0.07
0.1	0.11	0.11	0.22
1	0.35	0.35	0.7
10	1.11	1.11	2.21
100	3.50	3.50	7.00

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

13. EMC/Technical description

Electromagnetic environments		
Environment	Locations	General characteristics
Typical health care	Hospital, large clinic, doctor's office	Partly controlled, covered by the general requirements of this collateral standard
Residential	Doctor's office, small clinic	Not controlled, health care professional present
Residential	Home	Not controlled, health care professional not normally present
Transport, mobile	Car, aircraft (fixed-wing and helicopter), ambulance	Not controlled, wide variations, critical receivers nearby, harsh environments for ESD, RF, electric and magnetic fields
Special	Operating theatre, emergency room	Case-by-case examination of environment

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Medela AG