medela 

Instructions for use

**SpecialNeeds[®]
Feeder**

Manual de instrucciones

**SpecialNeeds[®]
Tetina**

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**SpecialNeeds[®]
Tetina**

SpecialNeeds® Feeder


Indication

The SpecialNeeds Feeder is suitable for use for babies with severe feeding problems such as cleft lip/palate, babies with the Pierre Robin Syndrome or Down Syndrome or for weak babies after an operation. This product should not be used for babies with a poor swallowing reflex. The Mini SpecialNeeds Feeder uses the same design; however, the teat is a smaller version for smaller or premature infants.

Important


Before use, separate the membrane from the valveplate. Wash and sterilize **all** parts, e.g. boiling for 3–5 min. or autoclaving. The teat is a wear and tear product and should be replaced when necessary.

Assembling Valve and Feeder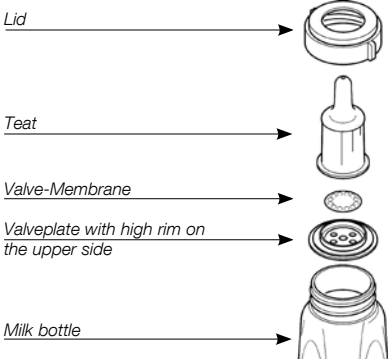

1. Press membrane onto valveplate (upper side), so that the stud goes completely through the central hole.
2. Put required amount of feed into the bottle.
3. Put assembled valve across the mouth of the bottle (make sure that the membrane and high rim of valveplate are upwards).
4. Place the teat over the valveplate and screw all the parts together with the lid.

Filling the teat

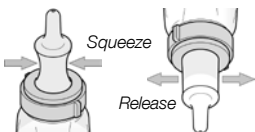1. Hold the feeder upright and *squeeze* some air out the teat.
2. *Keep squeezing* and turn the feeder upside down.

- Then *release* and some milk will enter the teat.
- Repeat several times until the teat is completely full.

Contents

The whole standard teat holds 29 ml

The whole mini teat holds 25 ml

The standard teat holds 6 ml

The mini teat holds 4 ml

0 ml

The contents of the bottle can be read on the scale up to 150 ml.

Controlling the Flow


The teat has an opening with a slit-valve, which can be adjusted to suit your baby. The flow rate is shown by the three lines on the teat, longest line for maximum flow, medium line for medium flow, shortest line for minimum flow.

Slit valve

a) Minimum flow

b) Medium flow

c) Maximum flow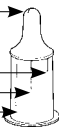

How the slit valve works

a) Slit horizontal

pressure keeps valve closed for *minimum flow*.

b) Slit diagonal

valve partly opens for *moderate flow*.

c) Slit vertical

pressure opens valve for *maximum flow*.


Use minimum flow if your baby needs to get used to the teat before feeding starts. The slit closes between sucks, so your baby is never flooded. To adjust the rate of flow, turn the feeder so that the required line (minimum, medium or maximum) points towards your baby's nose.

Example

For *medium flow*,

point medium line towards nose.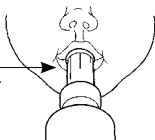

If necessary, you can help your baby to feed

Gently squeeze and release the teat to put a small amount of milk into your baby's mouth. Always wait for baby to swallow before squeezing again.

Cleaning and Sterilisation

Before first use and once daily

The disassembled parts of the feeder should be boiled for 5 minutes. In order to reduce lime deposits (which may influence the proper functioning of the valve), we recommend to use distilled water. All parts are suitable for cleaning in the dishwasher.

When used in hospitals, all disassembled parts can be autoclaved at 134° C for 3 (or 18) min. or 125° C for 15 min. To avoid damage during sterilisation, sterilise parts without applying external pressure on parts.

After each use

We recommend to clean the SpecialNeeds Feeder after each use. If there is some milk left in the teat, hold it nearly horizontal over a sink before unscrewing the lid. The valve should remain in the teat, to prevent milk from spilling. Take the SpecialNeeds Feeder apart, making sure that the valve is separated from the teat. Wash all parts thoroughly in hot soapy water and rinse with cold water. Let the parts air-dry on a dry, clean towel.

Material

Teat and valve-membrane: Silicone
Rest: Polypropylene

Disposal

According to local regulations.

CE-mark according to 93/42/EEC

Now available from Medela

«Give us a little time», a beautiful booklet on breast-feeding cleft lip/palate babies.

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